NISSKA®

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Please read the following instructions carefully before use.

The NISSKA® lice and nit comb is a medical device and allows the thorough removal of head lice and lice eggs (nits), which stick to the hair, by way of its narrowly spaced teeth. Freshly deposited nits are 1 to 2 mm away from the scalp and can only be removed by combing directly at the root of the hair. Egg cases 1 cm away from the scalp are usually empty and white because the larvae have already hatched. They are easier to recognize than the unhatched brownish lice eggs.

Wet combing with the NISSKA® comb

1. Prepare accessories such as hair conditioner, paper and hand towels, hair clips, styling comb and hand brush. Lay a bright towel over the shoulders to better recognize fallen head lice.
2. Wet hair with water. Generously apply a conditioner to wet hair. Comb the hair thoroughly with a styling comb in order to remove knots and to avoid later discomfort from the nit comb. Wipe any combed out head lice on a cloth and kill them immediately.
3. Divide the head into four hair sections. Part from the forehead to the nape of the neck and from ear to ear. Secure with hair clips and work through one after the other. Separate a lock of hair, that is narrower than the NISSKA® comb, and hold it tightly between your fingertips. Gently position the nit comb directly against the scalp, keeping the hair as close to the handle of the comb as thickness of the hair allows, and comb down through the lock of hair to the ends.
4. Regularly wipe the NISSKA® comb on a towel. Remove nits stuck in the comb with a hand brush under running water.
5. Repeatedly comb through strand by strand until no lice and nits can be seen. In particular check the areas behind the ears, the nape of the neck and the hairline. Make sure that the hair remains moist and that treated hair is not mixed with untreated hair. Finally, rinse the hair with water.

Successful treatment requires time and patience.

The Robert Koch Institute recommends a treatment every four days, so on the 1st, 5th, 9th, 13th and a follow-up on the 17th day. During the first day primarily adult lice are removed, the follow-up treatments are used to remove hatched larvae before they become sexually mature and lay new eggs.

Cleaning

Clean the nit comb under running water with a hand brush, brushing from the handle toward the tips of the teeth. If necessary, place the NISSKA® comb in boiling water for 1 minute to disinfect it.

Warnings

Use the NISSKA® comb with great caution. Due to the narrow spacing of the teeth, hair may fall out during combing. In case of head rash or lesions consult a doctor before use. Keep the NISSKA® comb out of reach of children. It should only be used on humans for its intended purpose.

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